



USER GUIDE AND
WASHING GUIDE



DEFENDER &
DEFENDER PRO

POWERTRAVELLER




DEFENDER

- 1. Wash your hands thoroughly before putting the mask on.**
- 2. Place each elastic around the ear. Adjust the mask so that it provides a tight but comfortable fit around your nose and mouth by adjusting the strap.**
- 3. Put on and remove your mask carefully. Avoid touching your eyes, nose and mouth when removing the mask. Wash your hands immediately after removing.**

STEP 1



STEP 2

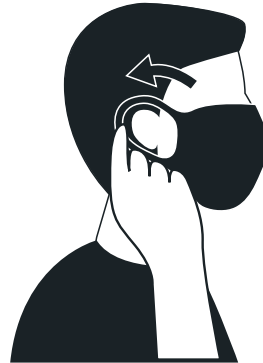

STEP 3




DEFENDER PRO

1. Wash your hands thoroughly before putting the mask on.
2. Insert the filter in the pocket inside the mask.
3. Place each elastic around the ear. Adjust the mask so that it provides a tight but comfortable fit around your nose and mouth by adjusting the strap.
4. Put on and remove your mask carefully. Avoid touching your eyes, nose and mouth when removing the mask. Wash your hands immediately after removing.


STEP 1



STEP 2

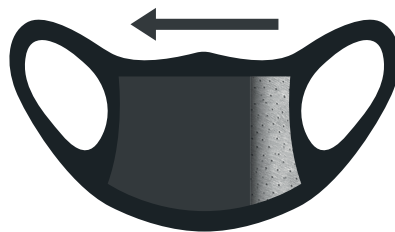

STEP 3

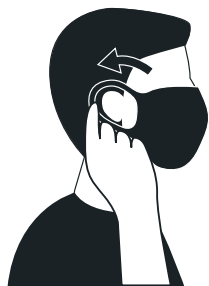

STEP 4



WASHING INSTRUCTIONS

It is strongly advisable to wash the mask with hand & detergent and hang to dry to prolong the product's life.

The masks will stand up to 40 - 50 machine washes depending on the cycle type .

Use cool water temperature, gentle cycle, at lower spin. Use non-chlorine bleach only.

Tumble dry low. Do not iron.

As with any other garment or textile product, if discoloration occurs or the mask becomes uncomfortable, you should replace the mask.

Washing the filter are not recommend as machine wash will diminish 95% filtration capacity of the filter. You may sanitize the filter after each use.

For further information please consult CDC instructions on “How to Wash Cloth Face Coverings”:

<https://www.cdc.gov/coronavirus/2019-ncov/prevent-getting-sick/how-to-wash-cloth-face-coverings.html>

WARNING

The masks are not surgical masks and are not intended to provide liquid barrier protection.

Masks are not certified for use in medical settings. Consult your supervisor prior to use in a medical setting.

Masks are not NIOSH certified N95 respirator

Misuse may cause injury or death

Wearing the mask will block particulates, including those carrying various viruses, but we cannot guarantee against contraction of any infections or diseases.

Time use limitation: Replace the mask and/or the filter when it becomes damaged or difficult to breathe through.

DEFENDER PRO SPECIFICATIONS:

- 95% filtration efficiency against 0.3 micron or larger particulates
- MERV16 Rating
- Three layer lightweight composite filter: Spunbound non-woven Polypropylene/Nanofiber/Non-woven Polyester
- Mask dimensions with filter: 8" x 6.3" (20.3cm x 16cm)
- Replaceable filter: Light design 0.04"/1mm thickness & 2.06-2.36 oz/yd² (70-80 g/m²) weight
- Replaceable filter: Air permeability of 28-35 cfm
- Replaceable filter designed for comfort and breathability: (air flow resistance) Pressure Drop <60 Pa

DEFENDER SPECIFICATIONS:

- Face-seal mask providing a tight fit around nose and mouth
- Made in the USA
- Designed to provide comfort and ease of breathing
- Effective filtration through double layer
- Washable and reusable
- Outer layer of mask, made from Spandex

DEFENDER PRO BLACK

Part Code: PPE-BDP001
Ean Code: 5060176985070

DEFENDER PRO GRAY

Part Code: PPE-GDP001
Ean Code: 5060176985100

DEFENDER BLACK

Part Code: PPE-BDF001
Ean Code: 5060176985018

DEFENDER GRAY

Part Code: PPE-GDF001
Ean Code: 5060176985049